

Online CBCT Viewer Access

LOGIN to the O-RAD website.

Click 'View Reports' tab and login to your report viewer.

Click on the SEARCH tab.

You can search your referred patients by using their name, Patient ID or Accession number.

Surname, Firstname

Searching for patients with compound names?

My patients

Click on your patient's name on the patient list. This will open up their study (see below).

Accession Number	Exam Description	Exam Date	Radiologist	Order Status	Organization	Image Availability
1234567	CT CBCT AXIAL REPORT	2021-Feb-15 11:58	Dr. John. Smith	Order Complete	WMBL	ONLINE
1234567	CT CBCT AXIAL REPORT	2021-Feb-15 11:58	Dr. John. Smith	Order Complete	WMBL	ONLINE

Double click on the scan with image availability 'ONLINE'

Surname, Firstname

DOB: 1962-Dec-20 (58 Y) Gender: Male Patient ID: 1

Accession Number	Exam Description	Exam Date	Radiologist	Order Status	Organization	Image Availability
	CT	2021-Feb-15 11:58 PM	---	Unknown (UNKNOWN)	WMBL	ONLINE
	CT CB REPORT	2021-Feb-15 01:39 PM	---	Order Complete (CM)	KARR	NO IMAGES

Double click on the fine slice Axial CBCT data to open the CBCT viewer window



CT
(400 images)

In the top toolbar, click on 'Open in advanced visualisation mode' to switch to MPR Mode to view in 3 planes.

